
RLS GLOBAL

CARISOLV® INSTRUCTIONS FOR USE


GB	Instructions for use	4
DE	Gebrauchsanweisungen	6
ES	Instrucciones de uso	9
PT	Instruções de utilização	12
FR	Mode d'emploi	15
IT	Istruzioni per l'uso	18
SE	Bruksanvisning	20
NO	Bruksanvisning	23
DK	Brugsanvisning	25
NL	Instructies voor gebruik	28
PL	Instrukcja użytkowania żelu	30
RO	Instrucțiuni de utilizare	33


1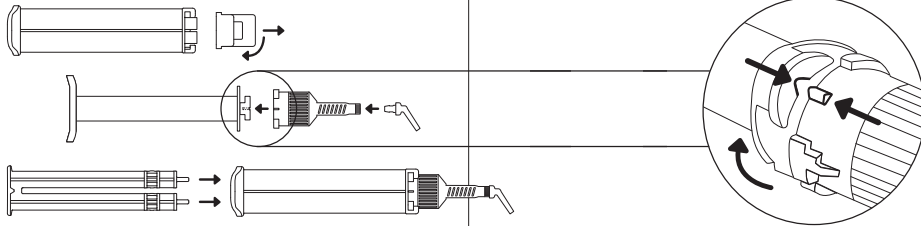


2

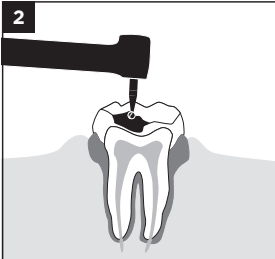
3

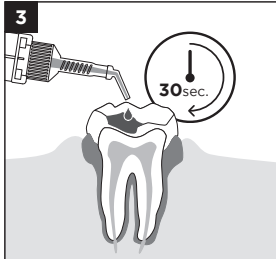
4

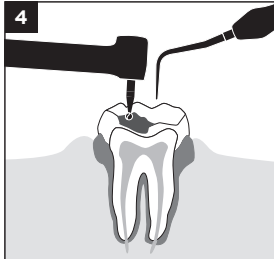
5

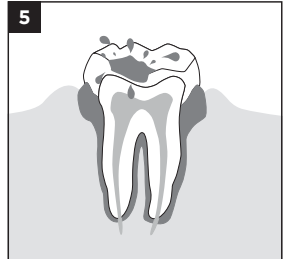6

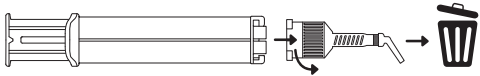7

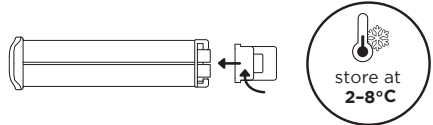GB

CARISOLV® CARIES REMOVAL GEL

Carisolv shall be used by qualified dental professionals.

INTENDED USE/PURPOSE

Softens demineralized carious dentine prior to mechanical removal during chemo- mechanical minimal invasive treatment of patients diagnosed with caries.

PRODUCT DESCRIPTION

Carisolv consists of a multiple-use dental dispenser i.e. a pre-filled syringe, plunger, disposable static mixers and intra-oral application tips. Carisolv softens carious demineralised dentine during the treatment which can be removed with the use of a sharp edge or blunt end cutting angle instrument leaving healthy tissue intact. Drilling may be applied e.g. for penetrating the enamel, improvement of the view or for better accessibility to the cavity for certain types of caries or for removal of old fillings.

Carisolv consists of two solutions provided in a double syringe with a closable cap. The solutions are mixed to form a gel at the time of use. Each package consists of 2 syringes packaged in aluminium pouches, 2 plungers, 10 static mixers and 10 intra-oral tips. The mixing of the solutions forms a clear gel, with a slight chlorine odour. The mixed gel is applied directly to the cavity and is left to act. The carious dentine can be removed followed by rinsing and filling according to general procedures.

INSTRUCTIONS FOR USE

Carisolv treatment

- 1 Remove the cap and fit the static mixer, the intra oral tip and the plunger to the syringe. Apply a disposable barrier sleeve to the syringe.
- 2 If necessary, open the cavity using standard dental burs. Prepare the outer contour free from caries, focusing on the enamel-dentine border of the cavity

and remove any remaining filling which is to be discarded.

- 3 Discard the first drops of the gel and apply the gel directly into the cavity with the intra-oral tip. Leave the gel to act for 30 seconds. Make sure that the carious lesion is thoroughly soaked with the gel.
- 4 Remove the carious dentine with the use of hand excavators or with ceramic minimally invasive burs as needed. For larger/deeper cavities, repeat the procedure from step 3-5.
- 5 Rinse the cavity with water and blow dry for inspection. Check carefully for carious dentine at the enamel-dentine border. The surface may appear matt and rough.
- 6 Apply traditional filling therapy. The static mixer and tip shall be removed and discarded when the treatment is completed.
- 7 Seal the syringe with the end cap directly after use and keep refrigerated during storage (2-8°C).

INGREDIENTS

A clear, viscous solution consisting of water, carboxymethyl cellulose, sodium hypochlorite, sodium chloride, and amino acids (glutamic acid, leucine, lysine) with alkaline pH.

PRECAUTIONS

- Carisolv shall not be used if the patient is suspected to be allergic to any of the ingredients.
- The static mixer and the intra-oral tip are single patient single use devices and must not be re-used. If re-used, there is risk for cross-contamination.
- To avoid contact of the multiple-use dental dispenser/syringe with the patient's mouth, apply a disposable barrier sleeve before use with each patient.
- Use new, uncontaminated gloves when handling the multiple-use dental dispenser/syringe.
- Shall not be injected.
- Carisolv shall not be used together with chlorhexidine.

- In case of contact with the eyes, rinse carefully with plenty of water for several minutes.

HANDLING, STORAGE & DISPOSAL

Product shall be used within one month from opening the pouch.

Keep the product refrigerated (2-8°C) in its original packaging during storage.

Disposal shall be handled in accordance with local regulations and instructions.

Can stain clothing.

SHELF-LIFE

Use prior to the expiry date as found on the package label.

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RLS Global AB